homecare

JOYCE Full Face vented

JOYCE Full Face vented 40 hPa

JOYCE Full Face non vented 40 hPa

JOYCE Full Face GEL vented

JOYCE Full Face GEL non vented

JOYCE Full Face^{plus} vented

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1 Description of Equipment

Legend

(Representation of individual parts: see Fig. A in enclosed leaflet.)

- 1 Headgear
- 2 Forehead cushion
- 3 Fine adjustment component of forehead support
- 4 Coarse adjustment component of forehead support
- 5 Forehead support
- 6 Headgear clip
- 7 Pressure measurement ports
- 8 Port cap
- 9 Anti-asphyxia valve (vented variants only)
- 10 Rotating sleeve*
- 11 Elbow*
- 12 Retaining ring*
- 13 Mask frame
- 14 Mask cushion
- 15 Chin strap (JOYCE Full Face^{plus} vented only)

*vented variants: transparent;
non vented variants: blue

Intended use

The JOYCE Full Face masks are accessories. They act as a sealing element between the patient and the therapy unit. They are used for the treatment of sleep apnea and for the non-invasive and non-life-sustaining ventilation of patients with ventilatory insufficiency.

The non vented variants may only be used in combination with ventilation devices which have an active exhaled air valve.

Before use by the patient

Patients must be instructed in the use of the mask by qualified personnel before therapy begins.

2 Safety instructions

Safety regulations

Following the requirements of Directive 93/42 ECC please observe the following points:

- Please read these operating instructions through carefully. They are a part of the product and should be available at all times.
- Never use the vented variants if the anti-asphyxia valve is damaged or missing, as this may lead to increased CO₂ reinhalation if the therapy unit fails.
- Never use JOYCE Full Face masks for patients who are not able to remove it independently or have a tendency to vomit. There is a risk of asphyxia.
- Use the JOYCE Full Face masks only in the therapy pressure ranges intended for the individual variants:
 - 4-30 hPa: JOYCE Full Face vented
JOYCE Full Face^{plus} vented
 - 4-40 hPa: JOYCE Full Face vented 40 hPa
JOYCE Full Face non vented 40 hPa
JOYCE Full Face GEL vented
JOYCE Full Face GEL non ventedOtherwise it can lead to CO₂ reinhalation or to leakages.
- If third-party items are used, functional failures may occur and fitness for use may be restricted. Biocompatibility requirements may also not be met. Please note that in such cases, any claim under warranty and liability will be voided if neither the accessories nor genuine replacement parts recommended in the instructions for use are used.
- Patients with limited spontaneous respiration must be monitored at all times. The appropriate negative pressure/leak alarm on the ventilation device must be activated.
- Patients with obstructive and restrictive lung diseases should be examined on an individual basis during use of the JOYCE

Full Face masks (e.g. with regard to the reinhalation of CO₂).

- Never close the opening of the anti-asphyxia valve or the exhaled air system. Otherwise there is a risk of reinhaling CO₂ and of asphyxia.
- Do not place the mask over your face for a prolonged period unless the therapy unit is switched on. Otherwise there is the risk of reinhaling CO₂.
- Do not use a breathing hose that is electrically conductive (exception: breathing tubes which can be heated electrically).
- Follow the instructions in section 4 "Hygiene treatment" to avoid infection, bacterial contamination and poor functionality of the mask.

Contraindications:

Under certain circumstances, JOYCE Full Face masks should not be used or only be used with particular care. In each case the decision whether to use these masks for therapy lies with the doctor in charge.

- Functional disturbance of the lower oesophageal sphincter (lower sphincter of the oesophagus)
- Excessive regurgitation (reflux of stomach contents into the oesophagus)
- Cough reflex disturbance
- Hiatus hernia (diaphragmatic hernia)
- Anesthetic or sedation
- Open facial wounds which tend towards inflammation
- Haemodynamic or cardiorespiratory instability
- Unconsciousness
- Claustrophobia, anxiety
- Facial deformation or deformation of the end of the nasopharynx
- Beard or other obstructions to adjusting and sealing the mask
- Facial injuries
- Barotraumas

- Necessity for ventilation or artificial respiration 24 hours a day
- Fresh facial, oesophageal or stomach operations
- Taking medication that can cause vomiting
- Necessity for immediate intubation

Side effects

The following side effects may occur:

Nasal congestion, dry nose, dry mouth in the morning, sinus pressure, chest pains, breathlessness, flatulence, headaches, irritation of the conjunctiva, irritating noises during exhalation, pressure sores on the face, reddening of the skin.

3 Operation

Fig. A-I: See enclosed leaflet.

3.1 Adjust forehead support (Fig. C)

Fine adjustment component

Squeeze the upper side tabs of the coarse adjustment component inwards. Position the fine adjustment component in one of the three slots.

To adjust the height of the forehead cushion or the fine adjustment component (approximately 1 cm), turn the fine adjustment component through 180°. Then position this as described above.

Coarse adjustment component

Squeeze the lower side tabs of the coarse adjustment component inwards. Place them in one of the three slots in the mask frame.

3.2 Put on the mask (Fig. B)

Caution!

To minimize the risk of vomiting, the patient should not eat or drink anything for 3 hours before using the mask.

1. Unhook one of the lower headgear clips from the mask.

- Place the headgear over your head and hook the headgear clip back onto the strap.
- Position the mask on your face as shown in Figure B.

Note:

If using JOYCE Full Face^{plus} vented, ensure that the chin strap is correctly located, i.e. that it surrounds your chin completely. Otherwise the mask may slip when you move.

- Adjust the headgear with the aid of the hook-and-loop closures so that the mask seals on the face but is not too tight.

WEINMANNstrap only:

The WEINMANNstrap headgear provides the following additional options (see also "Overview" on the enclosed leaflet):

- hook-and-loop closures on both sides.
- hook-and-loop closure on top of the head: release the hook-and-loop closure, pull gently on the ends of the headgear with both hands and push the hook-and-loop closure back on.

You can tell whether the headgear is correctly attached by the hook-and-loop tabs. The hook-and-loop tabs on the forehead are the same color as the inside of the headgear.

HEADstrap only:

The HEADstrap headgear provides the following options for you to do this:

- coarse adjustments: slot of the headgear.
- fine adjustments: four hook-and-loop straps."

- If necessary, re-adjust the forehead support.

3.3 Exhaled air system (Fig. H)

vented variants

vented variants (all mask components are transparent) have an **integrated exhaled air system**: the retaining ring and mask

frame are shaped in such a way that a gap is formed between these parts. Through this gap the exhaled air can escape. The special geometry of the parts guarantees that when putting on the retaining ring the gap cannot be closed. **Do not** connect an external exhaled air system, otherwise the therapy pressure will not be reached.

non vented variants

Warning!

The non vented variants (retaining ring, elbow and rotating sleeve in blue) **do not have their own exhalation system!** They may only be used in combination with therapy devices which have an **active exhalation valve!** If the therapy device fails, there is otherwise the risk of re-inhalation of CO₂ and suffocation.

The non vented variants may **never** be used in combination with an **external exhalation system** (such as Silentflow2, for example), otherwise the safety function is not guaranteed in the event of device failure. See also the instructions for use for your therapy device.

3.4 Anti-asphyxia valve (Fig. G)

Warning!

Only the vented variants are equipped with an anti-asphyxia valve. Ensure that this valve is working correctly **before every use** (see "5. Functional check" on page 21), otherwise there is a **risk of suffocation!**

Safety function of the anti-asphyxia valve

The anti-asphyxia valve has an opening to the atmosphere through which the patient can inhale ambient air if the therapy device fails. This significantly reduces the risk of re-inhaling CO₂ and thus also the risk of suffocation.

Inside the valve is a membrane which can adopt either of two positions:

- As long as the therapy device is in operation, the membrane is pressed against the opening to the atmosphere by the flow,

sealing this opening completely. The patient breathes through the therapy device.

- If the therapy device fails or if the device is not connected, the membrane is in the resting position, in other words, the opening to the atmosphere is free. The patient breathes ambient air through this opening.

Caution!

The anti-asphyxia valve is a highly sensitive component. Always handle it with care.

3.5 Gel-filled mask components

On JOYCE Full Face GEL vented and JOYCE Full Face GEL non vented, the forehead cushion and the mask cushion are gel-filled as standard. Gel forehead cushions and gel mask cushions are also available as replacement parts for other JOYCE Full Face variants.

Caution!

Gel-filled mask components are highly sensitive components. Always handle them with care to prevent deformation and damage.

3.6 Mask cushions with chin strap

JOYCE Full Face^{plus} vented has a chin strap which supports the chin during therapy to stop the lower jaw slipping out of the mask cushion and interrupting therapy.

3.7 Therapy pressure ranges

The individual JOYCE Full Face mask variants are each available for a particular therapy pressure range (see "7. Scope of supply" on page 22):

- 4-30 hPa: JOYCE Full Face vented
JOYCE Full Face^{plus} vented
- 4-40 hPa: JOYCE Full Face vented 40 hPa
JOYCE Full Face non vented 40 hPa
JOYCE Full Face GEL vented
JOYCE Full Face GEL non vented

The JOYCE Full Face masks for the therapy pressure range 4 - 30 hPa have a soft mask cushion, the JOYCE Full Face masks for the therapy range 4 - 40 hPa have a harder mask

cushion or a gel mask cushion (gel-filled variants).

The harder mask cushion for the therapy pressure range 4 - 40 hPa is marked with the number "40".

All mask cushions are also available as individual replacement parts to enable you to adapt your mask to changed therapy requirements.

3.8 Connecting the mask to the therapy unit (Fig. H)

Push the hose system of your therapy unit onto the rotating sleeve. Ensure that the connection to the hose is secure and airtight. The connection must not be loosened by the therapy pressure.

If you want to detach the mask temporarily from the hose, pull the rotating sleeve off the elbow.

Caution!

If there is a high risk of transferring bacteria between the patient and the unit, a bacterial filter should be used. Also pay attention to the instructions for use of the therapy unit.

3.9 Ports for pressure measurement (Fig. E)

JOYCE Full Face masks are fitted with two ports that can be used to measure pressure or to feed in oxygen.

Caution!

- Oxygen therapy can have side effects if incorrectly administered or used in the wrong dosage. Only carry out this sort of treatment if prescribed by a doctor.
- When carrying out oxygen therapy, strictly follow the instructions for use of your O₂ system and your therapy unit.

When the pressure measurement ports are not in use, always close them with the caps provided. Otherwise air will escape from the ports and the therapy pressure will not be reached.

3.10 Compatible devices and accessories

Therapy units

JOYCE Full Face masks can be combined with all units designed for carrying out the therapies described under "Intended Use".

Note that if unsuitable device combinations are used, the actual pressure in the mask may not correspond to the therapy pressure you have been prescribed.

This may also happen even if the correct pressure is indicated on the therapy unit. Have your device combination adjusted by a doctor or distributor so that the pressure in the mask corresponds to your therapy pressure.

Humidifier

JOYCE Full Face masks can be used with both cold and warm-air humidifiers.

Never set the humidification level so high that condensation forms in the air hose. Follow the instructions for use of your humidifier and therapy unit.

Quick release cord (optional) (Fig. I)

To open the headgear in an emergency (e.g. vomiting) a quick release cord can be supplied as an accessory.

The quick release cord consists of a special clip with ripcord, which is fitted instead of one of the lower headgear clips.

In an emergency, simply pull the ripcord. The special clip will detach itself from the mask. The headgear is now open and the mask can be removed.

Bronchoscopy set (optional) (no figure)

The bronchoscopy set (endoscopy adapter non-vented in combination with retaining ring non vented (NV)) allows endoscopy (bronchoscopy) to be performed without interrupting ventilation.

The bronchoscopy may only be performed by trained medical staff.

For more detailed information, please see brief instructions for use WM 67495.

3.11 After usage

1. Undo one of the lower headgear clips from the mask and remove the mask.
2. Undo the other three headgear clips in order to remove the headgear from the mask.
3. Proceed as described in "4 Hygiene treatment".

3.12 Disassembling/assembling the mask

Disassembling the mask (Fig. G - C)

1. Detach the mask cushion from the mask frame.
2. Remove the cap from the pressure measurement ports.
3. Turn the retaining ring anticlockwise to the 11 o'clock position. Now pull the retaining ring and the elbow away from the mask frame.
4. Press the elbow out of the retaining ring and pull the rotating sleeve off the elbow.
5. **Only vented variants:** Remove the anti-asphyxia valve carefully off the elbow. Ensure that the membrane on the interior of the valve is not damaged.
6. Detach the coarse adjustment component from the mask frame.
7. Detach the fine adjustment component from the coarse adjustment component. Then detach the forehead cushion from the fine adjustment component.

Assembling the mask (Fig. C - G)

1. Attach the forehead cushion to the fine adjustment component. Then attach the fine adjustment component to the coarse adjustment component.
2. Attach the coarse adjustment component to the mask frame.
3. Push the rotating sleeve onto the elbow until you hear it click into place.

4. **Only vented variants:** Push the anti-asphyxia valve carefully onto the elbow until you hear it click into place on both sides of the elbow. Take care that the membrane on the interior of the valve is not damaged or jammed.
5. Press the rotating sleeve onto the elbow. Then press the elbow into the retaining ring until you hear it click into place.
6. Push the retaining ring (in the 11 o'clock position) and elbow onto the mask frame. Turn the retaining ring to the 12 o'clock position.
7. Push the cap onto the pressure measurement ports.
8. Attach the mask cushion to the mask frame.
9. Fasten the headgear to the mask with the clips.
10. Check that everything is working properly (see "5. Functional check" on page 21).

4 Hygiene treatment

4.1 Intervals

Daily

Clean the mask in warm water to remove all residues. To do this the mask can be disassembled as described in section 3.12 "Disassembling/assembling the mask". Use a mild cleaning agent (e.g. dishwasher detergent). After cleaning, rinse all parts carefully with clean water.

Caution!

- For patients with a weakened immune system or a special medical history, **daily** disinfecting or sterilization of the mask components may be necessary. In this case, please consult your doctor.
- Please take great care when cleaning and disinfecting the contact surfaces between the mask frame and the retaining ring. Residues on these surfaces can reduce the

success of the therapy and the functioning of the integrated exhaled air system (vented variants).

Weekly

Disassemble the mask as described in section 3.12 "Disassembling/assembling the mask" and subject the parts of the mask to a hygiene treatment (see "4.2. Cleaning, disinfection, sterilization" on page 20).

4.2 Cleaning, disinfection, sterilization

Permissible procedures

Procedure	Dishwasher	Hand wash	Disinfection*	Sterilization (alternative to disinfection)
Mask cushion	•	•	•	•
Mask cushion GEL	•	•	•	
Forehead cushion	•	•	•	
Forehead cushion GEL		•	•	
Port cap	•	•	•	•
Mask frame	•	•	•	
Coarse adjustment component	•	•	•	
Fine adjustment component	•	•	•	
Elbow	•	•	•	
Anti-asphyxia valve		•	•	
Rotating sleeve	•	•	•	
Retaining ring	•	•	•	
Headgear clip	•	•	•	
Headgear WEINMANNstrap		•		
Headgear HEADstrap		•		
Headgear JOYCEstrap		•		
Quick release cord		•		

Caution!

- Always clean the components of the mask thoroughly before disinfecting or sterilizing them as indicated in the above table.
- The emergency exhaled air valve and gel-filled mask components are highly sensitive

components. Always handle them with care to prevent damage or deformation.

Description of procedures

Dishwasher	Rinse cycle at 65°C. Then carefully rinse all parts with clean water.
Hand wash	Hand wash in warm water. Use a mild detergent. Then rinse all parts in clean water and allow to air-dry.
Disinfection*	Clean mechanically, e.g. with a brush, in dilute solution. After disinfection rinse all parts thoroughly with distilled water and dry thoroughly.
Sterilization (alternative to disinfection)	Steam sterilization in devices to comply with EN 285. Temperature 134 °C, minimum hold time 3 minutes.

*We recommend the disinfectants gigasept® FF (new), Cidex® OPA and Aniosyme DD1. Evidence of tolerance to these disinfectants has been provided. It is essential to follow the manufacturer's instructions.

Notes on the hygiene treatment and on the hygiene treatment for a change of patient can be found in a brochure from our information portal at www.weinmann.de. If you do not have Internet access, contact Weinmann.

Note

Wear suitable gloves (e.g. disposable gloves) during the hygiene treatment.

Wash the HEADstrap headgear before using for the first time, as the colour might run.

The headgear may only be washed by hand!

Do not iron the headgear, otherwise the hook-and-loop closures will no longer work!

Do not dry the headgear in a dryer!

4.3 Following treatment

Thoroughly dry all parts following the hygiene treatment. Check the components for residual contamination or damage. Repeat the hygiene treatment if required. Replace damaged parts. Reassemble the mask.

4.4 Multiple patient use

If the mask is used by a different patient, it must be subjected to a hygiene treatment first.

Disassemble the mask. Disinfect or sterilize all parts as described in section "4.2 Cleaning, disinfection, sterilization".

Use a new headgear and where necessary a new quick release cord.

5 Functional check

Before use check that everything is working properly as described below.

vented variants

Always check that the anti-asphyxia valve is working before each use as described below:

1. Ensure that the valve and membrane are dry, undamaged and free of secretions. Do not use the mask if the valve or membrane is damaged (e.g. through tears and deformation). In this case replace the anti-asphyxia valve. (see "7. Scope of supply" on page 22)
2. Ensure that the valve membrane is not closing the valve aperture. Only then can ambient air flow into the mask.
3. Attach the breathing hose (delivered with the therapy unit) to the elbow of the mask and to the therapy unit.
4. Switch on the therapy unit. The membrane should now close the aperture of the anti-asphyxia valve. Only then can ambient air flow from the therapy unit into the mask.
5. Do not use the mask if the valve or the membrane is not working. Replace the anti-asphyxia valve. (see "7. Scope of supply" on page 22)

JOYCE Full Face masks

Carry out a visual check after each cleaning and disinfecting procedure.

If the mask cushion is damaged and is no longer airtight, replace it with a new one.

Check the functionality and air-tightness of the ball-and-socket joint on the elbow. If it no longer moves easily or is no longer airtight, replace the elbow and retaining ring.

Parts that are no longer usable can be disposed of as normal household waste.

The forehead cushion and the mask cushion may become discolored, but this does not impair function.

6 Troubleshooting

Mal-functions	Cause	Solution
Painful pressure on the face.	Mask is too tight.	Loosen headgear slightly. Adjust position of forehead support.
Draught in the eyes.	Mask is too loose.	Tighten headgear slightly. Adjust position of forehead support.
	Mask does not fit.	Use mask of different size. Available from your distributor.
	High therapy pressure.	Use hard mask cushion (for pressures up to 40 hPa). Available from your distributor.
Therapy pressure is not reached.	Mask not adjusted correctly.	Adjust headgear. Adjust position of forehead support.
	Mask cushion is damaged.	Replace mask cushion.
	Hose system is not airtight.	Check push-fit connector and fit of hose sleeves.
	Air is escaping from pressure measurement ports.	Cover pressure measurement ports with cap (included in package).
	Anti-asphyxia valve: – incorrectly fitted – defective	Anti-asphyxia valve: fit correctly replace

7 Scope of supply

Note

All masks are supplied fully assembled and with headgear.

vented variants

Article	Article no. with WEINMANNstrap/ HEADstrap	Article no. with JOYCEstrap
JOYCE Full Face vented, size S	WM 26410	WM 26410/1
JOYCE Full Face vented, size M	WM 26420	WM 26420/1
JOYCE Full Face vented, size L	WM 26430	WM 26430/1
JOYCE Full Face vented, size XL	WM 26440	WM 26440/1
JOYCE Full Face vented 40 hPa, size S	WM 26411	WM 26411/1
JOYCE Full Face vented 40 hPa, size M	WM 26421	WM 26421/1
JOYCE Full Face vented 40 hPa, size L	WM 26431	WM 26431/1
JOYCE Full Face vented 40 hPa, size XL	WM 26441	WM 26441/1
JOYCE Full Face GEL vented, size S	WM 26412	WM 26412/1
JOYCE Full Face GEL vented, size M	WM 26422	WM 26422/1
JOYCE Full Face GEL vented, size L	WM 26432	WM 26432/1
JOYCE Full Face ^{plus} vented, size S	WM 26413	
JOYCE Full Face ^{plus} vented, size M	WM 26423	
JOYCE Full Face ^{plus} vented, size L	WM 26433	
JOYCE Full Face ^{plus} vented, size XL	WM 26443	

non vented variants

Article	Article no. with WEINMANNstrap/ HEADstrap
JOYCE Full Face non vented 40 hPa, size S	WM 26461
JOYCE Full Face non vented 40 hPa, size M	WM 26471

Article	Article no. with WEINMANNstrap/ HEADstrap
JOYCE Full Face non vented 40 hPa, size L	WM 26481
JOYCE Full Face non vented 40 hPa, size XL	WM 26491
JOYCE Full Face GEL non vented, size S	WM 26462
JOYCE Full Face GEL non vented, size M	WM 26472
JOYCE Full Face GEL non vented, size L	WM 26482

Replacement parts

Article	Article no.
Ball-and-socket joint set: Retaining ring, elbow, rotating sleeve, anti-asphyxia valve	WM 26569
Ball-and-socket joint set non vented (blue): Retaining ring, elbow, rotating sleeve	WM 26207
Rotating sleeve vented	WM 26254
Forehead support: Forehead cushion, coarse adjustment component forehead support, fine adjustment component forehead support	WM 26201
Forehead support GEL: Forehead cushion GEL, forehead support coarse adjustment component, forehead support fine adjustment component	WM 26115
Forehead cushion	WM 26200
Forehead cushion GEL	WM 26209
Mask cushion 30 hPa, size S	WM 26510
Mask cushion 30 hPa, size M	WM 26520
Mask cushion 30 hPa, size L	WM 26530
Mask cushion 30 hPa, size XL	WM 26540
Mask cushion 40 hPa, size S	WM 26511
Mask cushion 40 hPa, size M	WM 26521
Mask cushion 40 hPa, size L	WM 26531
Mask cushion 40 hPa, size XL	WM 26541
Mask cushion GEL FFM, Size S	WM 26512
Mask cushion GEL FFM, Size M	WM 26522
Mask cushion GEL FFM, Size L	WM 26532
Mask cushion FF ^{plus} , size S	WM 26586
Mask cushion FF ^{plus} , size M	WM 26587
Mask cushion FF ^{plus} , size L	WM 26588
Mask cushion FF ^{plus} , size XL	WM 26589
WEINMANNstrap headgear, incl. headgear clips	WM 26350

Article	Article no.
Headgear HEADstrap, inc. headgear clips	WM 26360
JOYCEstrap headgear, blue, incl. headgear clips	WM 26290
Headgear clips (4 pcs.)	WM 26205
Port cap	WM 26208
Instructions for use DE, EN, FR, NL, IT, ES, PT, TR, EL, PL, HR	WM 66117

Accessories

Article	Article no.
Quick release cord (ripcord)	WM 26560

8 Specifications

Technical data

Parameters	Value
Product category according to 93/42/EEC:	II a
Dimensions (WxHxD):	c. 107 x 160 x 106 mm ¹
Weight:	c. 102 grams ¹
Therapy pressure range: Mask cushion 30 hPa: Mask cushion 40 hPa: Mask cushion GEL FFM: Mask cushion FF ^{plus} :	4 to 30 hPa 4 to 40 hPa 4 to 40 hPa 4 to 30 hPa
Hose connector: cone complying with EN ISO 5356-1 vented variants: non vented variants:	Ø 22 mm (male) Ø 22 mm (female)
Pressure measurement ports:	Ø 4 mm
Width of headgear clips:	max. 20 mm
Temperature range Operation: Storage:	+ 5° C to + 40° C - 20° C to + 70° C
Flow resistance vented variants: at 50 l/min at 100 l/min non vented variants: at 50 l/min at 100 l/min	0.02 hPa 0.26 hPa 0.09 hPa 0.37 hPa
Flow resistance anti-asphyxia valve Inspiration at 50 l/min.: Exhalation at 50 l/min.:	0.5 hPa 0.5 hPa

Parameters	Value
Switching pressure anti-asphyxia valve (vented variants) Open: Close:	≤ 1 hPa ≤ 2 hPa
Quoted two-figure noise emission value to ISO 4871: sound pressure level sound power level (uncertainty factor: 3 dB(A))	16.5 dB(A) 24.5 dB(A)
Working life	up to 12 months ²
Standards applied	EN ISO 17510-2:2009

¹ depending on size and design

² If the materials used to produce masks are exposed to aggressive cleaning agents, for example, they age. In individual cases, therefore, you may need to replace your mask sooner (see "5. Functional check" on page 21).

CE 0197 Subject to design changes

Materials


Article	Material
Mask cushion, port cap	Silicone
Mask cushion GEL	Silicone, silicone gel
Forehead cushion, retaining ring	TPE, Polypropylene
Forehead cushion GEL	Silicone, silicone gel, polycarbonate
Mask frame, elbow	Polycarbonate
Headgear WEINMANNstrap	Lyra, Polyester, Polyurethane, Nylon, Cotton, Polyamide
Headgear JOYCEstrap	Cotton, polyamide, elasthane
Headgear HEADstrap	Nylon/Spandex Polyurethane Polyester/Lyca
Rotating sleeve, forehead support coarse adjustment component, forehead support fine adjustment component	Polypropylene
Headgear clip	POM
Valve cartridge (anti-asphyxia valve)	TPE, Polypropylene
Quick release cord (ripcord)	POM, Polyester, Polyamide

Article	Material
Plug for pressure measurement connections	Silicone
Plug for endoscopy adapter non-vented	Silicone
Mask cushion, forehead cushion	Silicone and silicone gel, silicone coating
Mask body, elbow connector, endoscopy adapter non-vented	Polycarbonate, Polyamide
Retaining ring, emergency exhalation valve	Polypropylene, TPE

All mask components are latex-free.

Pressure/flow characteristic curve

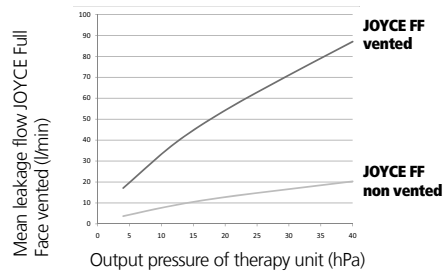The pressure/flow characteristic curve shows outlet flow as a function of therapy pressure.

9 Warranty

Weinmann gives the customer a limited manufacturer warranty on new genuine Weinmann products and any replacement part fitted by Weinmann in accordance with the warranty conditions applicable to the product in question and in accordance with the warranty periods from date of purchase as listed below. The warranty conditions can be downloaded from www.weinmann.de on the Internet. We can also send you the warranty conditions on request. In the event of a claim under warranty, please contact your specialist dealer.

Product	Period of guarantee
Weinmann devices including accessories (except masks) for sleep diagnosis, sleep therapy, home ventilation, oxygen medicine and emergency medicine	2 years
Masks including accessories, rechargeable batteries, batteries*, sensors, tube systems	6 months
Disposable products	None

*unless there is information to the contrary in the technical documentation.

10 Declaration of conformity

Weinmann Geräte für Medizin GmbH + Co. KG declares herewith that the product complies fully with the respective regulations of the Medical Device Directive 93/42/EEC.

The unabridged text of the Declaration of Conformity can be found on our website at www.weinmann.de

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