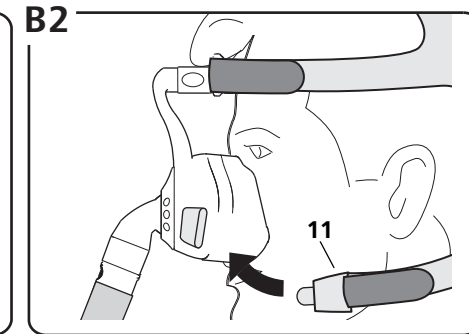
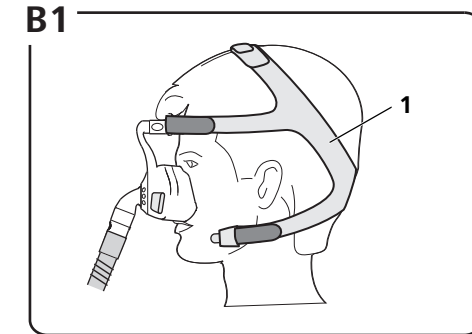
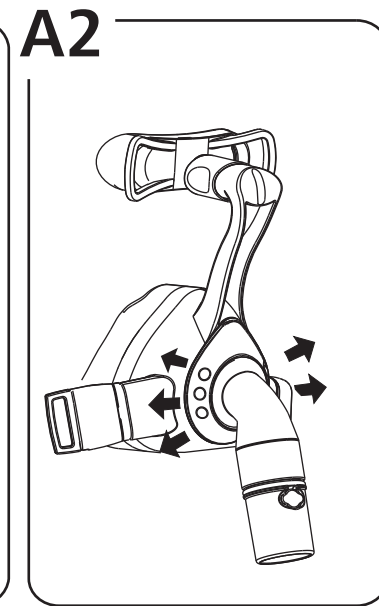
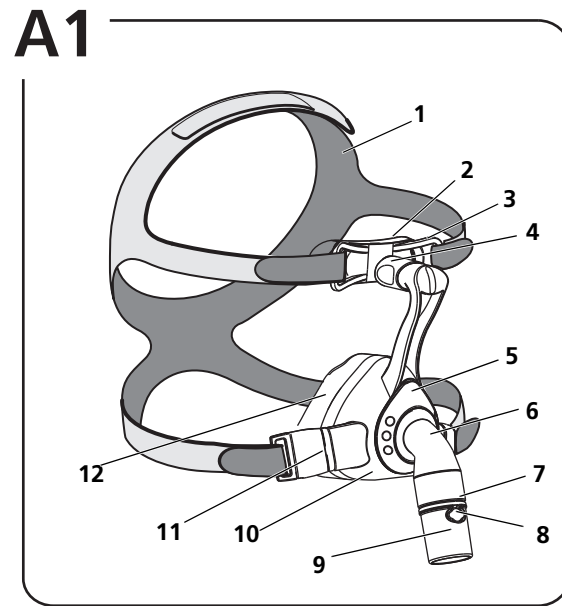
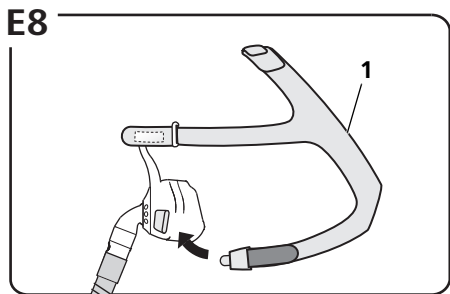
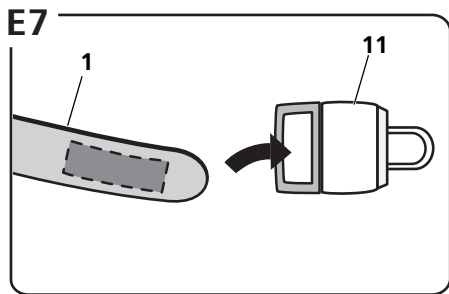
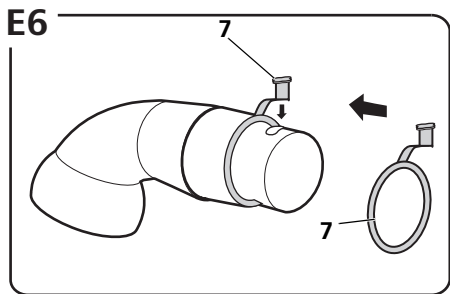
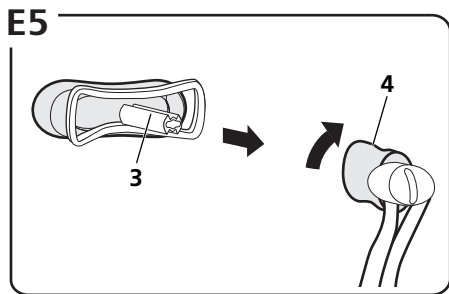
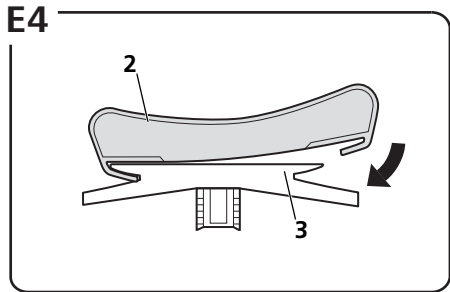
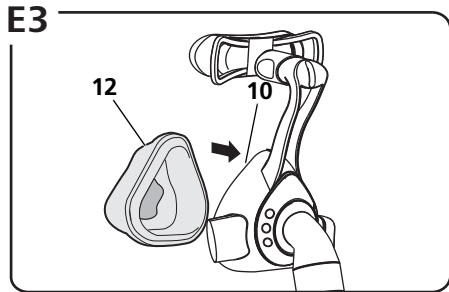
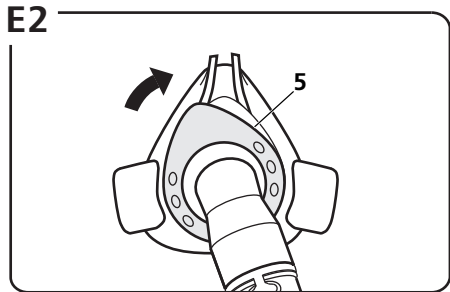
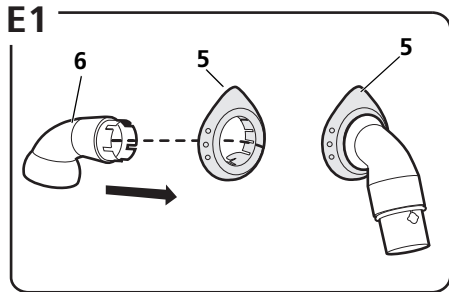
homecare

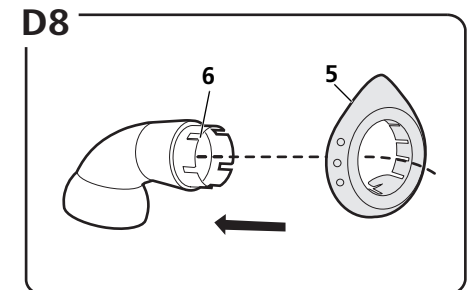
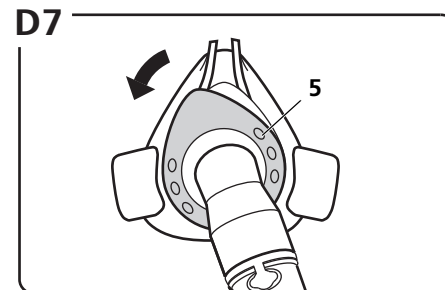
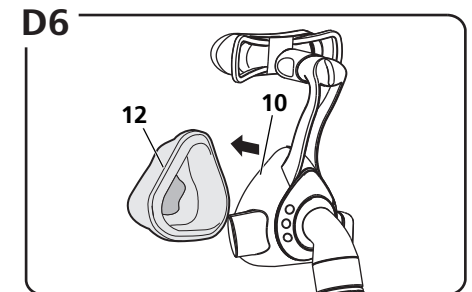
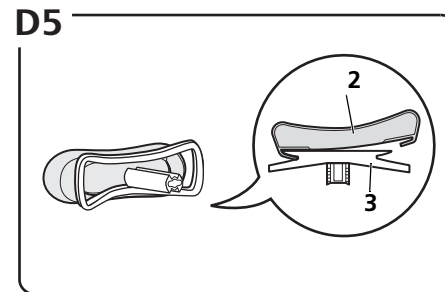
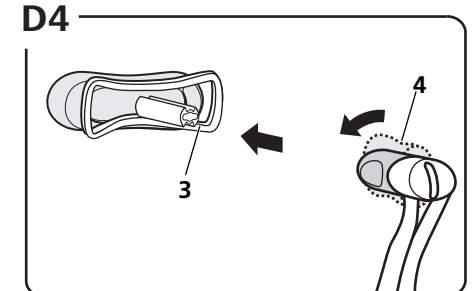
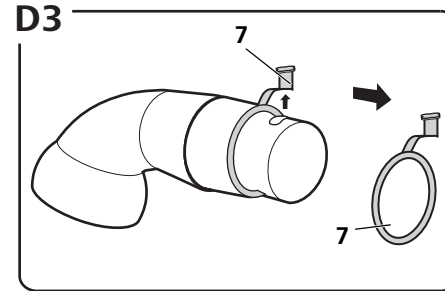
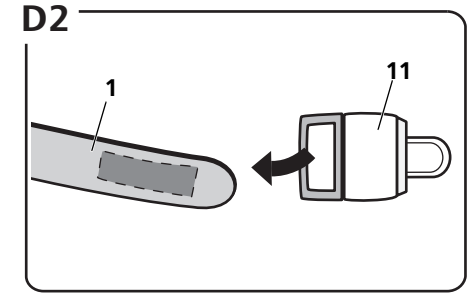
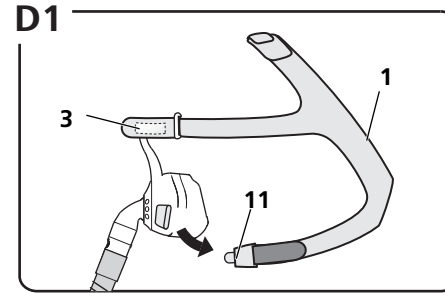
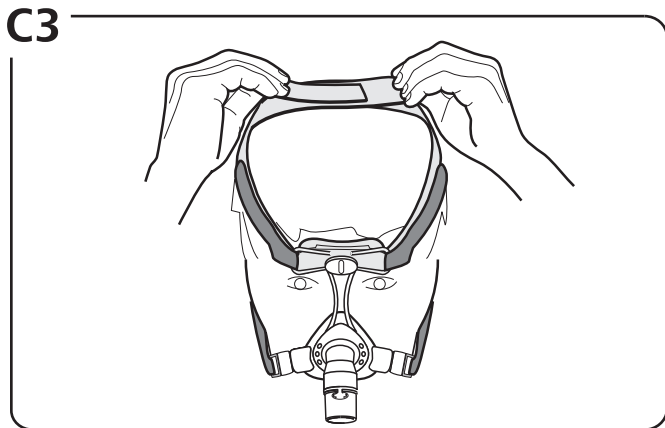
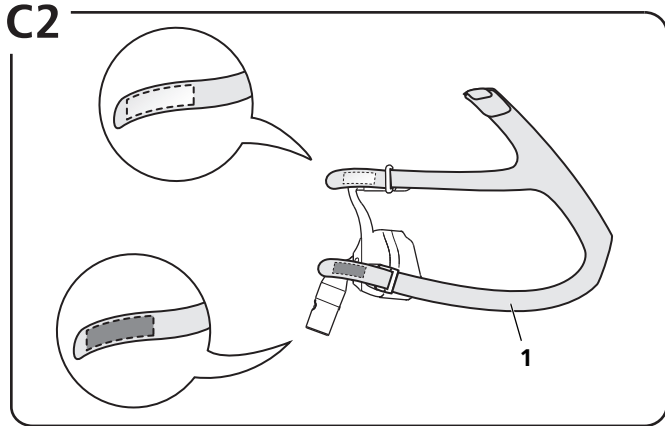
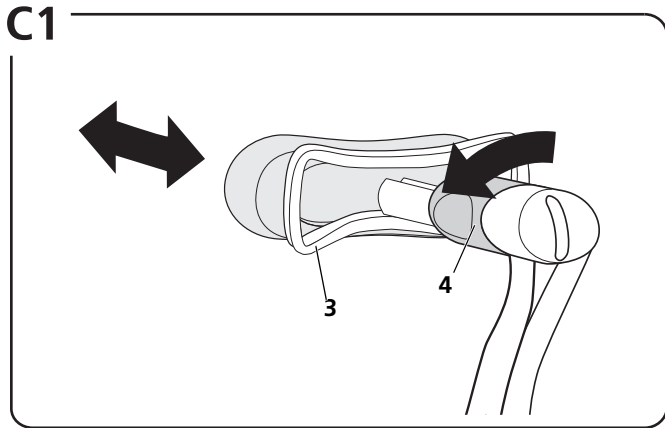
JOYCEeasy

Nasal Mask

Instructions for Use

WEINMANN
medical technology

Deutsch	3
English	12
Français	21
Nederlands	31
Italiano	41
Español	51
Русский	61
Türkçe	71
Ελληνικά	80
中文	90

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Table of Contents

1 Introduction	13	9 Technical data	18
1.1 Intended use	13	9.1 Materials	19
1.2 Contraindications	13	9.2 Characteristic pressure/ flow curve	19
1.3 Side effects	13	10 Warranty	19
2 Safety	13	11 Declaration of conformity	20
2.1 Safety information	13		
2.2 General information	14		
3 Product description	14		
3.1 Overview	14		
3.2 Compatible devices	14		
3.3 Exhalation system	15		
3.4 Pressure measurement port ...	15		
4 Operation	15		
4.1 Applying the mask	15		
4.2 Adjusting the mask	15		
4.3 Removing the mask	15		
4.4 Dismantling the mask	15		
4.5 Assembling the mask	16		
5 Hygiene preparation	16		
5.1 Cleaning the mask	16		
5.2 Tips	17		
5.3 Change of patient	17		
6 Disposal	17		
7 Faults	17		
8 Scope of supply	18		

1 Introduction

1.1 Intended use

This nasal mask JOYCEeasy is used for treating sleep apnea and for non-invasive ventilation of patients with respiratory insufficiency. It serves as a connecting element between the patient and the therapy device.

1.2 Contraindications

In the case that any of the following symptoms are present the mask may not be used, or only with particular caution:

erosions and ulcerations, skin allergies, flush of the facial skin, pressure marks on the face, claustrophobia, anxiety, deformities of the face or nasopharynx, intake of medication which may induce vomiting, the necessity for immediate intubation. Please also observe the contraindications in the instructions for use of your therapy device.

1.3 Side effects

Nasal congestion, dry nose, morning xerostomia (dry mouth), feeling of pressure in the sinuses, irritated conjunctiva, skin rashes, pressure marks in the face, irritating noises when breathing.

2 Safety

Please read these instructions carefully. They are a constituent part of the mask and must be available at all times. In accordance with Directive 93/42/EEC, observe the points below.

2.1 Safety information

Risk of injury from the supply of oxygen!

Oxygen can deposit on clothing, bed linen and hair. In conjunction with smoking, naked flames or electrical equipment, it can cause fires and explosions.

- ⇒ Do not smoke.
- ⇒ Avoid naked flames.
- ⇒ Do not use electrically conductive tubes (exception: breathing tubes which can be heated electrically).

Risk of injury from oxygen therapy!

Oxygen therapies can lead to side effects in case of incorrect use or dosage.

- ⇒ Only administer oxygen as prescribed by a physician.
- ⇒ Follow the instructions for use of the oxygen system and therapy device.

Risk of injury from re-inhaling CO₂!

If the mask is used incorrectly, CO₂ may be re-inhaled.

- ⇒ Do not close the openings of the mask.
- ⇒ Only apply the mask for a prolonged period if the therapy device is running.
- ⇒ Only use the mask within the quoted therapy pressure range.
- ⇒ Do not use the mask on patients who cannot take it off themselves.

Risk of injury from slipping of the mask!

If the mask slips or falls off, the therapy is ineffective.

- ⇒ Monitor patients with restricted spontaneous respiration.
- ⇒ Activate low pressure/leakage alarms on the therapy device.
- ⇒ Monitor patients with obstructive and restrictive lung diseases individually.

Patient endangerment through application of mask during anesthesia!

When mask is used during anesthesia, the supplied anesthetic gas can escape through the mouth and endanger the patient.

- ⇒ Do not use nasal mask during anesthesia.

2.2 General information

- Use of third-party products may lead to functional failures and restricted fitness for purpose. Biocompatibility may also be compromised. Please note that in these cases, any claim under warranty and liability will be void if neither the accessories nor original replacement parts recommended in the instructions are used.
- To prevent infection, bacterial contamination or functional impairments, follow the instructions in the section "Hygiene preparation" (see "5 Hygiene preparation", page 16).

3 Product description

3.1 Overview

Illustrations of the individual components can be found on the enclosed folding sheet (**figure A1**).

Key

- 1** Headgear
- 2** Forehead cushion
- 3** Forehead support
- 4** Lock
- 5** Retaining ring
- 6** Elbow
- 7** Plug for pressure measurement port
- 8** Pressure measurement port
- 9** Rotating sleeve
- 10** Mask body
- 11** Headgear clip
- 12** Mask cushion

3.2 Compatible devices

The mask can be used with all therapy devices which are not for life-sustaining treatment of patients with sleep apnea or ventilatory insufficiency.

With many combinations of device, the actual pressure in the mask will not correspond to the prescribed therapy pressure, even if the therapy device is displaying the correct pressure. Have the combination of devices adjusted by a physician or specialist dealer so that the actual pressure in the mask corresponds to therapy pressure.

3.3 Exhalation system

The mask has an integrated exhalation system. The retaining ring and mask body are shaped in such a way that a gap forms between these parts. The exhaled air can escape through this gap (**figure A2**).

3.4 Pressure measurement port

The mask has a connection **8** which can be used for measuring pressure or supplying oxygen. If you are not using the pressure measurement port, close it with the plug **7** to reach therapy pressure.

4 Operation

4.1 Applying the mask

1. Pull the headgear **1** over the head and place the mask on the face (**figure B1**).
2. Insert headgear clips **11** (**figure B2**).

4.2 Adjusting the mask

1. Undo lock **4** (**figure C1**).
2. Pull the forehead support **3** in or out of the lock **4** bit by bit as necessary (**figure C1**).

Remember the most convenient position. This way, the forehead support does not need to be adjusted after every disassembly.

3. To adjust the forehead support **3** to fit higher or lower on the forehead rotate the forehead support by 180°.
4. Applying the mask (see "4.1 Applying the mask", page 15).
5. Adjust headgear **1** with the aid of the hook-and-loop fasteners so that the mask seals on the face but is not too tight (**figure C2**).
6. To adjust the hook-and-loop fastener on top of the head undo the hook-and-loop fastener, pull on the ends of the headgear with both hands and close the fastener again (**figure C3**).

4.3 Removing the mask

1. Undo **11** the headgear clips.
2. Slip the headgear **1** over the head and remove the mask.

4.4 Dismantling the mask

1. Undo the headgear clips **11** (**figure D1**).
2. Release hook-and-loop fasteners from headgear clips **11** and forehead support **3** and remove the headgear **1** (**figure D1 + D2**).
3. Remove the plug for the pressure measurement port **7** (**figure D3**).
4. Open the lock **4** and remove the forehead support **3** (**figure D4**).
5. Remove forehead cushion **2** from forehead support **3** (**figure D5**).

6. Take mask cushion **12** off mask body **10** (**figure D6**).
7. Turn retaining ring **5** counterclockwise and remove it (**figure D7**).
8. Remove elbow **6** from retaining ring **5** (**figure D8**).

4.5 Assembling the mask

1. Insert elbow **6** in the retaining ring **5** (**figure E1**).
2. Insert retaining ring in the mask body with position **5** at 11 o'clock and turn clockwise to a 12 o'clock position until it locks into place (**figure E2**).
3. Fit mask cushion **12** onto mask body **10** (**figure E3**).
4. Pull forehead cushion **2** over the forehead support **3** (**figure E4**).
5. Insert the forehead support **3** in the lock **4** and close the **4** lock (**figure E5**).
6. Fit the plug for the pressure measurement port **7** (**figure E6**).
7. Attach the headgear **1** with the headgear clips using the lower hook-and-loop fasteners **11** (**figure E7**).
8. Attach the headgear **1** to the mask (**figure E8**).

You can tell whether the headgear is correctly attached by the hook-and-loop tabs. The hook-and-loop tabs on the forehead have the same color as the inside of the head bands.

5 Hygiene preparation

WARNING

Risk of injury from inadequate cleaning!

Residues may congest the mask, impair the integrated exhalation system and jeopardize therapy success.

⇒ For patients with a compromised immune system or particular background of illness, disinfect mask components daily following consultation with the physician.

⇒ Thoroughly clean the contact surfaces between the elbow and the rotating sleeve.

- **This product may contain disposable items. Disposable items are intended for single use.** Therefore, use these items only once and do **not** reprocess them. Reprocessing disposable items may impair functionality and safety of the product and lead to unforeseeable reactions due to ageing, embrittlement, wear, thermal stress, the effects of chemical processes etc.

5.1 Cleaning the mask

1. Dismantle the mask (see "4.4 Dismantling the mask", page 15).
2. Clean mask in accordance with the table below:

Action	Daily	Weekly
Wash mask components with warm water and a mild detergent.	X	
When washing mask parts (except for mask cushion and forehead cushion), clean thoroughly with a cloth or a soft brush.		X
Wash headgear by hand.		X

- Rinse all parts with clear water.

NOTICE

Material damage due to abrasion!

Rubbing the mask cushion and forehead cushion can damage the coating.

⇒ Do not rub the mask cushion and forehead cushion dry.

- Allow all parts to air-dry.
- Perform a visual inspection.
- Replace damaged parts if necessary.
- Assembling the mask (see "4.5 Assembling the mask", page 16).

Discolorations of mask components do not impair the functionality of the mask.

5.2 Tips

- You can wash all parts (except the headgear) in a dishwasher at 65 °C.
- Do not dry parts in direct sunlight.
- Wash headgear before first use, as the color may run.

- Do not iron the headgear in order to keep the hook-and-loop fasteners intact.
- Do not dry the headgear in a tumble-dryer.

5.3 Change of patient

You must subject the mask to a hygiene preparation in the event of a change of patient. For information on hygiene preparation in the event of a change of patient, please refer to our internet brochure at www.weinmann.de. If you do not have Internet access, please contact Weinmann.

6 Disposal

You can dispose of all parts in domestic waste.

7 Faults

Fault	Cause	Remedy
Facial pressure pain.	Mask fits too tightly.	Loosen headgear. Adjust position of forehead support.
Draft in the eyes.	Mask fits too loosely.	Tighten headgear. Adjust position of forehead support.
	Mask does not fit.	Use different mask size. Contact your specialist retailer.

Fault	Cause	Remedy
Therapy pressure is not reached.	Mask is not adjusted correctly.	Re-adjust mask (see 4.2, p. 15).
	Mask cushion is damaged.	Replace mask cushion.
	Tube system is leaking.	Check connection and fit of tubes.
	Air is escaping from pressure measurement connection.	Close pressure measurement port with plug (included in scope of supply).

8 Scope of supply

Size	Variant	Item number
S	vented	WM 25510
M	vented	WM 25520
L	vented	WM 25530

You can order replacement parts separately if necessary. A current list of replacement parts can be ordered on the Internet at www.weinmann.de or via your specialist dealer.

9 Technical data

	Nasal cannula
Product class according to directive 93/42/EEC	Ila
Dimensions (W x H x D)	approx. 80 mm x 130 mm x 90 mm ¹
Weight	ca. 72 g ¹
Therapy pressure range	4 hPa - 30 hPa
Hose connection: Cone according to EN ISO 5356-1	Ø 22 mm (male)
Pressure measurement port	Ø 4 mm
Temperature range: Operation Storage	+5 °C to + 40 °C -20 °C to +70 °C
Flow resistance at 50 l/min at 100 l/min	0.05 hPa 0.28 hPa
Quoted two-figure noise emission value according to ISO 4871: -Sound pressure level -Sound power level, uncertainty factor:	18 dB(A) 23 dB(A) 3 dB(A)
Service life	up to 12 months ²
Standards applied	EN ISO 17510-2:2009

¹ depending on size and variant.

² The materials of the mask deteriorate if exposed to e.g. aggressive detergents. In individual cases, therefore, it may be necessary to replace mask parts sooner.

CE 0197

The right to make design modifications is reserved.


9.1 Materials

Mask part	Material
Plug for pressure measurement port, mask cushion, forehead cushion	Silicone, silicone coating
Retaining ring (hard component)	PP (polypropylene)
Retaining ring (soft component)	TPE (thermoplastic elastomer)
Headgear clip, lock	PA (polyamide), POM (polyoxymethylene)
Forehead support, rotating sleeve, mask body, elbow	PA (polyamide)
Headgear	Lycra, polyester, polyurethane, UBL loop, cotton, polyamide

All parts of the mask do not contain latex, PVC (polyvinylchloride) and DEHP (diethylhexyl phthalate).

9.2 Characteristic pressure/flow curve

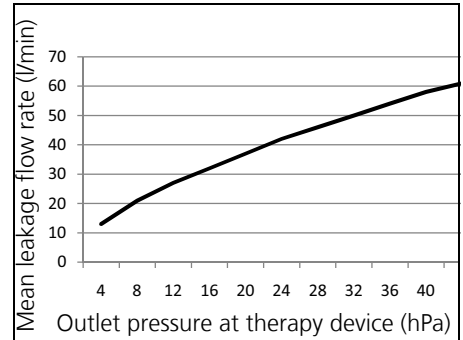The characteristic pressure/flow curve shows the outlet flow in dependence with therapy pressure.

10 Warranty

Weinmann gives the customer a limited manufacturer warranty on new genuine Weinmann products and any replacement part fitted by Weinmann in accordance with the warranty conditions applicable to the product in question and in accordance with the warranty periods from date of purchase as listed below. The warranty conditions can be downloaded from www.weinmann.de on the Internet. We can also send you the warranty conditions on request. In the event of a claim under warranty, please contact your specialist dealer.

Product	Period of guarantee
Weinmann devices including accessories (except masks) for sleep diagnosis, sleep therapy, home ventilation, oxygen medicine and emergency medicine	2 years
Masks including accessories, rechargeable batteries, batteries*, sensors, tube systems	6 months
Disposable products	None

*unless there is information to the contrary in the technical documentation.

11 Declaration of conformity

Weinmann Geräte für Medizin GmbH + Co. KG hereby declares that the product complies with the relevant regulations of Directive 93/42/EEC governing medical devices. The complete text of the declaration of conformity can be found at: www.weinmann.de

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Geräte für Medizin GmbH+Co.KG